




GV Health
Life. Protected.

Biohazard Spill Kit

DO NOT USE ON CHEMICAL OR URINE / VOMIT SPILLS



1
Read instructions before use. Put on all personal equipment (apron, gloves) and open clinical waste bag

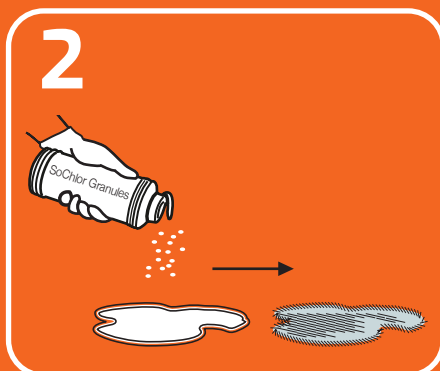

2
Pour SoChlor granules around the spillage perimeter, filling inwards. Wait 2-3 mins



3
Prepare 10,000ppm SoChlor disinfectant solution. Fill container with cold tap water to fill line and dissolve 7 SoChlor tablets



4
Collect granule absorbed spill with scoop & scraper and dispose of into clinical waste bag



5
Wet applicator with the SoChlor solution, wiping the contaminated area using an 'S' motion



6
Ensure all used components are disposed of into clinical waste bag. Rinse diluter and put back in kit

Tie bag securely and dispose of in accordance with ADR or local waste regulations



GV Health Ltd
Hall House, 2 Arlington Court, Whittle Way,
Arlington Business Park, Stevenage, Hertfordshire, SG1 2FS, UK
t +44(0)1920 463 098 e info@gvhealth.com
w www.gvhealth.com



Scan for
video guide

