


Environmental Hygiene & Disinfection

CLEAN AND DISINFECT IN ONE STEP WITH SOCHLOR DST

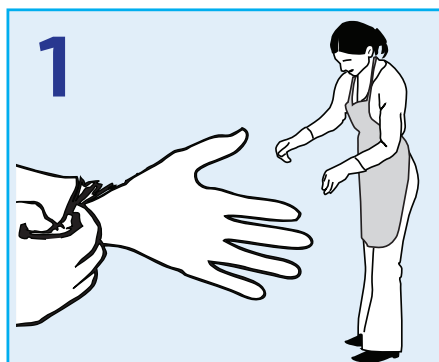


1
Wear disposable gloves and apron.
If risk of splashing wear eye protection.



2
Remove any gross contamination before
applying SoChlor[™], including urine & vomit.




3
Dissolve SoChlor[™] tablets (according to dilution
instructions below) in water. If solution is older
than 24hrs make new solution.

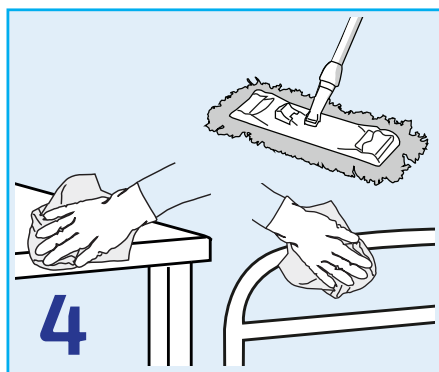

4
Use solution according to your hospital
policy. Once applied, allow surfaces to air dry.
(5 min contact time recommended)



5
Dispose of remaining solution into sluice
or drain with running water.

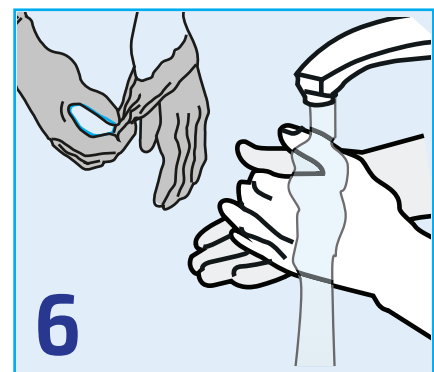

6
Wash your hands after removing gloves.

Dilution Instructions

Dilution rates for SoChlor DST per 1 litre of water		
Environmental Situation	Required amount of available chlorine	SoChlor DST / 1.7g NaDCC Tablet (MFB256 / MFB294)
Standard disinfection and clean	1,000 ppm	1
Outbreak disinfection	5,000 ppm	5
Disinfection of blood or blood-stained spills	10,000 ppm	10


Safety & Precautions



DON'T take
internally, avoid
eye and direct
skin contact.



DON'T mix
directly with
acids* or cationic
detergents
*eg. urine or vomit.

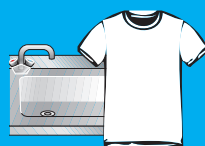

AVOID
PROLONGED
contact with
stainless steel
or clothing.



ALWAYS keep
out of the reach
of children &
vulnerable
patients.



ALWAYS dispose
used materials
as clinical waste.



ALWAYS make
fresh solution if
solution date is not
known.



ALWAYS replace
lid after use and
store in a secure
dry place.



WHERE
POSSIBLE
ensure good
ventilation.