


INFECTION PREVENTION

Healthcare to home



GV Health
Life. Protected.

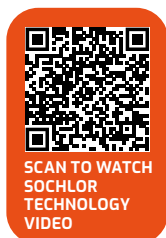

SoChlor[®] TAB

Dilution Guide

Use	Concentration (ppm)	Dilution per tablet size				Application
		0.5g	1.7g	2.5g	4.5g	
Vegetable / Salad wash	25 ppm	1 tablet in 12 litres	1 tablet in 40 litres	1 tablet in 60 litres	1 tablet in 100 litres	Wash Salad / Vegetables
Dishcloths, Mops	60 ppm	1 tablet in 5 litres	1 tablet in 16 litres	1 tablet in 25 litres	1 tablet in 40 litres	Soak to disinfect, clean and deodorise. Do not leave over night
Infant Bottles / Teats	125 ppm	1 tablet in 2.5 litres	1 tablet in 8 litres	1 tablet in 12 litres	1 tablet in 20 litres	Soak for 30 mins
Food preparation surfaces	200 ppm	1 tablet in 1.5 litres	1 tablet in 5 litres	1 tablet in 7.5 litres	1 tablet in 12 litres	Soak for 3 mins
WC's, Drains, Sinks	400 ppm	1 tablet in 750ml	1 tablet in 2.5 litres	1 tablet in 3.5 litres	1 tablet in 6 litres	Pour in quiet periods
Stainless Steel Instruments	600 ppm	2 tablets in 1 litre	3 tablets in 5 litres	2 tablets in 5 litres	1 tablet in 4 litres	Immerse for 1 hour
General Disinfection	1,000 ppm	4 tablets in 1 litre	1 tablet in 1 litre	2 tablets in 3 litres	1 tablet in 2.5 litres	Mop or wipe on hard & sealed surfaces
Laboratory discard jars	2,500 ppm	8 tablets in 1 litre	1 tablet in 400ml	5 tablets in 3 litres	5 tablets in 5 litres	Soak overnight
Bodily Fluids Disinfection	10,000 ppm	10 tablets in 300ml	1 tablet in 100ml	7 tablets in 1 litre	4 tablets in 1 litre	2 mins contact

Life. Protected.

gvhealth.com

Version: 110124